


■ CONTOURING AND INCREASING LIP VOLUME.
FILLING OF MEDIUM DEPTH WRINKLES

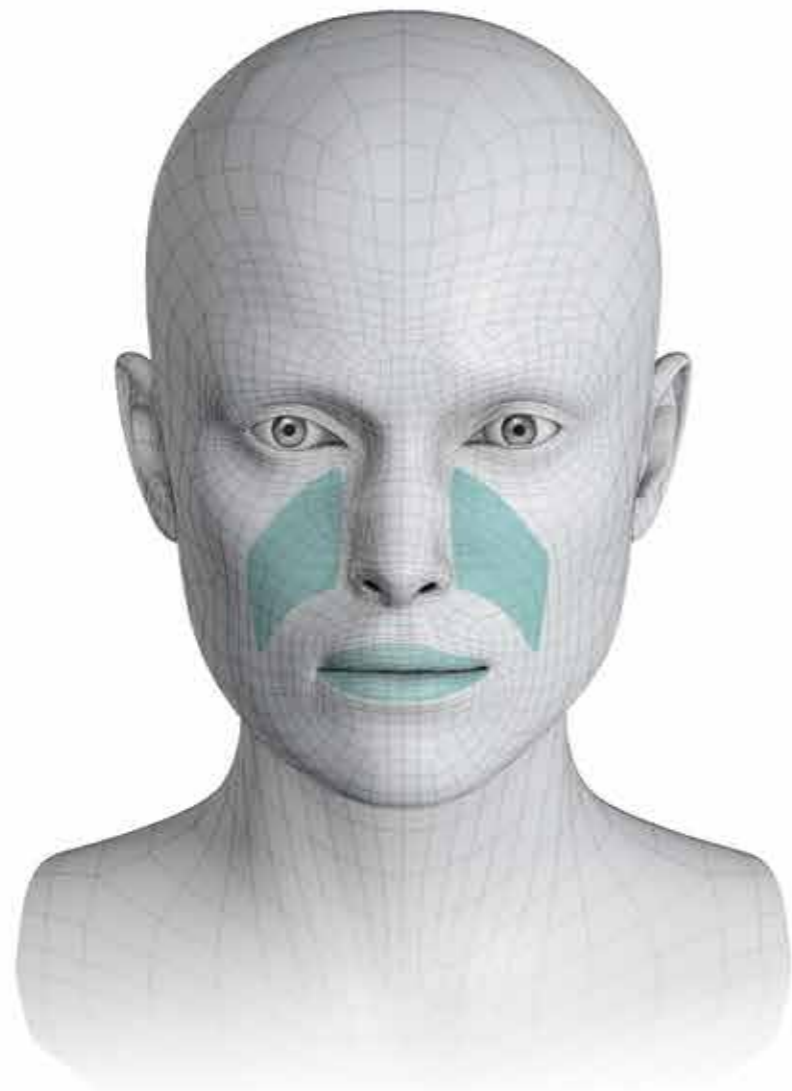

ALEXA MEDIUM¹

- Nasolacrimal furrow
- Nasolabial creases
- Lip volumizing
- Lip contouring


1. Application according to Alexa medium instruction. Recommendations for application of Alexa medium are approved by medical experts of Institute Hyalual company.

■ **Alexa MEDIUM**

High degree of satisfaction with procedures performed

Validated lip fullness scale (VLFGSV)


The image corresponds to validated Merz scale (Merz validated scale)

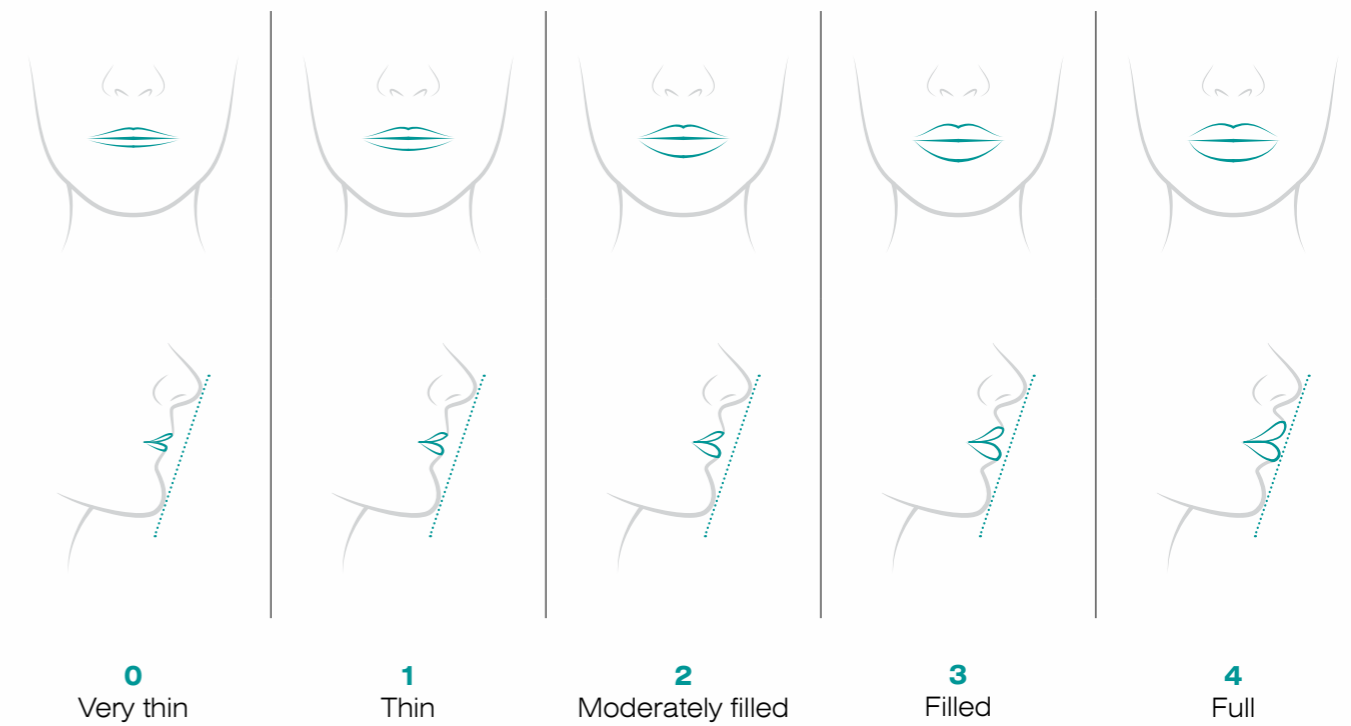


1. Clinical follow-up conducted in medical centers of Institute Hyalual. This follow-up is not of intervention nature. Alexa smooth/medium/volume injection implant was used in accordance with an official instruction. Internal data 2017.

Alexa MEDIUM


Высокая степень удовлетворенности от проведенных процедур

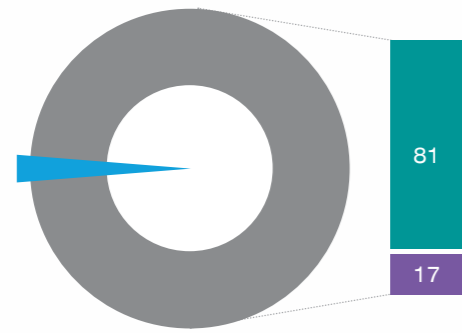

A Evaluation of VLFGSV after the procedure with Alexa medium
N=60

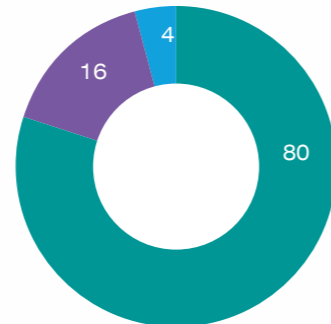

B Evaluation of VLFGSV 14 days after the procedure with Alexa medium
N=60

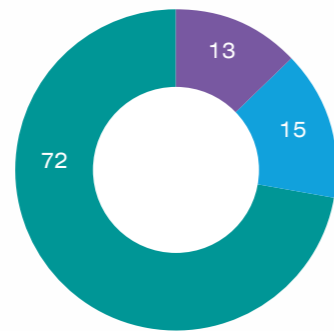

C Evaluation of VLFGSV 90 days after the procedure with Alexa medium
N=60

Very thin Thin Moderately filled Filled Full

Fig. (a), (b), (c) show percentages of follow-up participants with remaining correction results of Alexa medium 17.5 mg/ml application – initially (a), after 14 days (b), after 90 days (c)

RESULTS:

- 81% of patients received achievable lip volumizing «full lips» with Alexa medium 1 ml
- 17% of patients achieved «maximum lip fullness»
- In 85% of subjects doctors noted remaining results 90 days after the procedure

1. Clinical follow-up conducted in medical centers of Institute Hyalual. This follow-up is not of intervention nature. Alexa smooth/medium/volume injection implant was used in accordance with an official instruction. Internal data 2017.


Alexa MEDIUM

Proven clinical efficacy



Before the procedure

30 days after the procedure

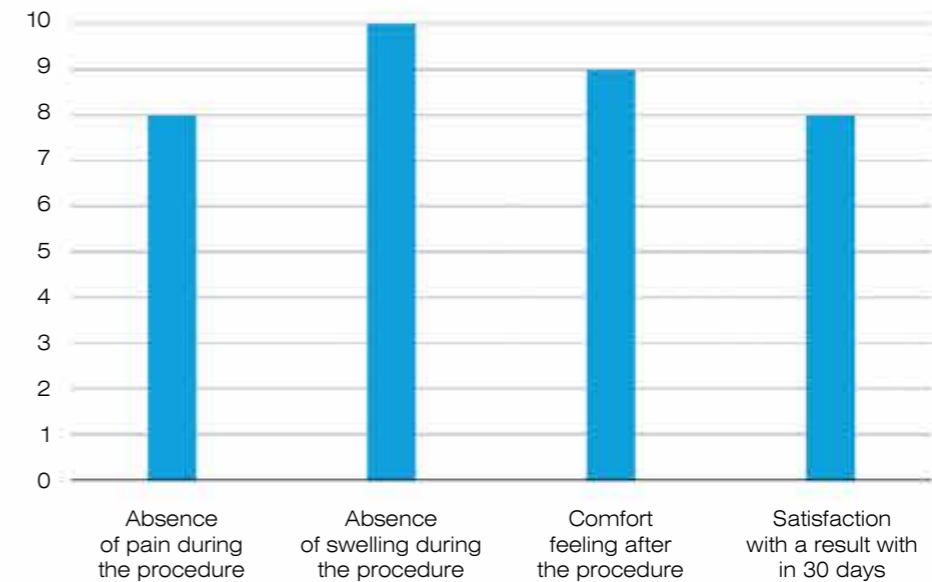

INSTITUTE HYALUAL EXPERT COMMENTS:

Correction:

application of Alexa medium. Alexa medium filler 0.6 ml was applied in to upper and lower lips

Results:

remaining of increased lip volume after 30 days


1. Internal data 2017. Results of clinical follow-up using Vectra software. 2. Data collected from patients' survey after correction with Alexa medium filler.

■ **Alexa MEDIUM**
Proven clinical efficacy



Before the procedure

60 days after the procedure

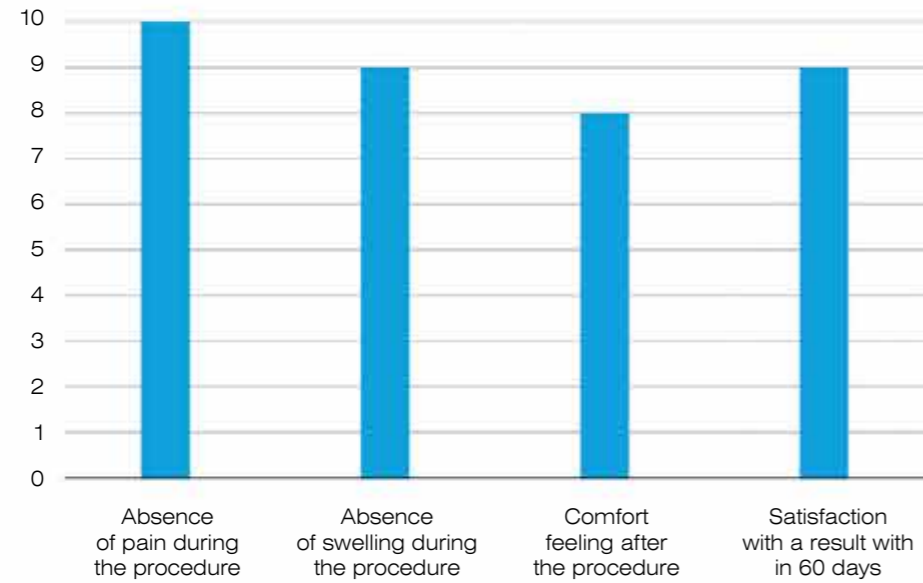

INSTITUTE HYALUAL EXPERT COMMENTS:

Correction:

application of Alexa medium. Alexa medium filler 1 ml was applied to upper and lower lips

Results:

remaining of increased lip volume after 60 days


1. Internal data 2017. Results of clinical follow-up using Vectra software. 2. Data collected from patients' survey after correction with Alexa medium filler.

■ **Alexa MEDIUM**
Proven clinical efficacy



Before the procedure

90 days after the procedure

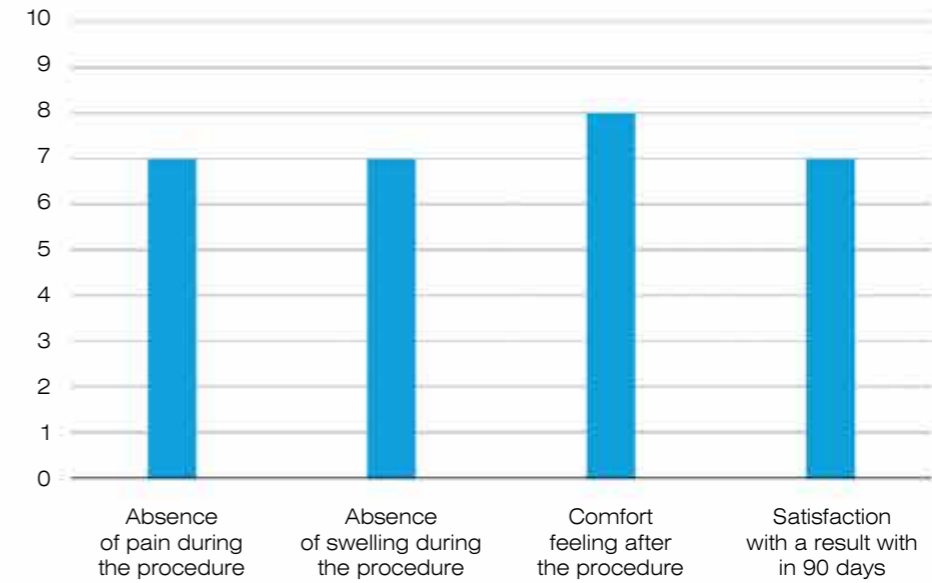

INSTITUTE HYALUAL EXPERT COMMENTS:

Correction:

application of Alexa medium. Alexa medium filler 1 ml was applied to upper and lower lips

Results:

remaining of increased lip volume by 25% after 90 days

1. Internal data 2017. Results of clinical follow-up using Vectra software. 2. Data collected from patients' survey after correction with Alexa medium filler.